


Attachable anywhere  
simply and securely with  
a sticker type.





How to use

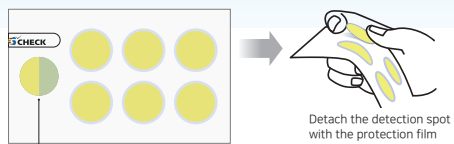

Positive test color




- ① Peel off the protective film before use.
- ② Place a drop of the liquid on the spot.
- ③ After 1 minute, if the color of the half part changes, DO NOT DRINK!!

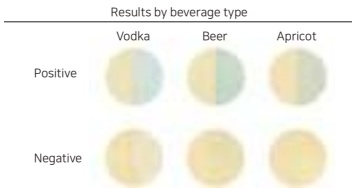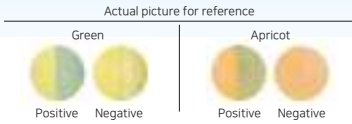

How to use



- ① Peel off the protective film before use.
- ② Place a drop of the liquid on the spot.
- ③ After 1 minute, check if the color changes.

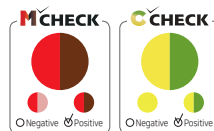


- ④ Mark the result.
- ⑤ Take off the whole sticker and attach it to documents.

Innovative  
Diagnostic  
POCT  
Platforms



**PHILMEDI**  
www.philmedi.com

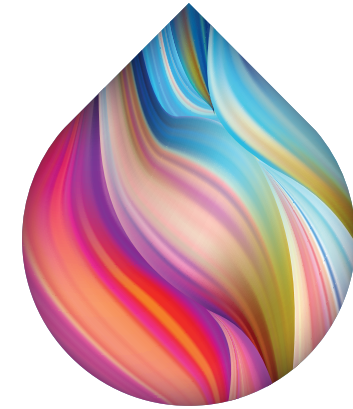

For the quick drug test,  
check easily right on spot now!

418 CenterM Building, 33, Sagimakgol-ro 62beon-gil, Jungwon-gu,  
Seongnam-si, Gyeonggi-do, Republic of Korea

T : +82-31-698-3112 E : marketing@philmedi.com  
H : www.philmedi.com



GHB without smell and taste which is used for sexual assault or date rape can be easily detected right on spot now.



G-CHECK is a sticker-type GHB detection paper designed to check whether GHB(Gamma-hydroxybutyric acid) is mixed in beverages or alcohol.




- Portable and easy to carry as small as business card in a size of 90x50mm
- Attachable anywhere with a sticker-type
- Quick test result from the product right on spot in just 1 minute
- No damage caused by change in temperature and humidity outside with the protective film

Methamphetamine  
Right on spot!  
In just 1 minute!



M-Check is a sticker-type METH detection paper designed to detect whether or not unknown material is real methamphetamine without additional devices.

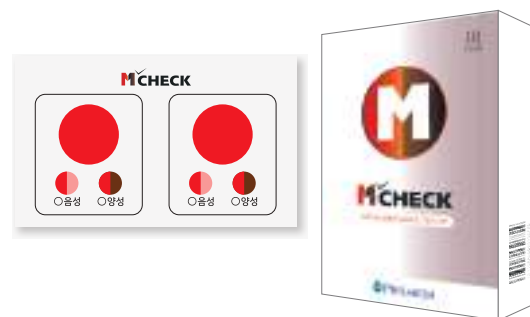


- Portable and easy to carry as small as business card in a size of 90x50mm
- Quick test result from the product right on spot
- No damage caused by change in temperature and humidity outside with the protective film
- Simple to report cases by attaching the result to official documents with a sticker-type

Cocaine can be detected simply and immediately right on spot!



C-CHECK is a sticker-type cocaine detection paper designed to detect whether or not unknown material is real cocaine without additional devices.

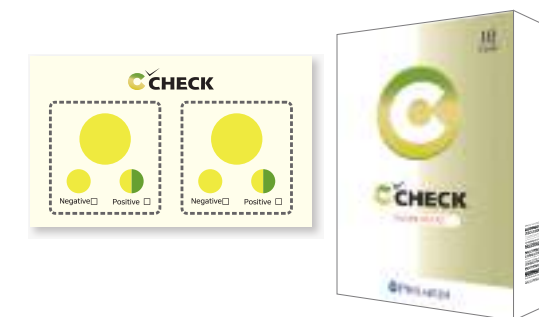

- Portable and easy to carry as small as business card in a size of 90x50mm
- Quick test result from the product right on spot
- No damage caused by change in temperature and humidity outside with the protective film
- Simple to report cases by attaching the result to official documents with a sticker-type